

Handset Functions

Hand Control

The hand control is equipped with four functions:

Backrest Section Adjustment (1)

Up: Press left hand button
Down: Press right hand button

Leg Support Section Adjustment (2)


Up: Press left hand button
Down: Press right hand button

Backrest and Leg Support Simultaneous Adjustment (3)

Up: Press left hand button
Down: Press right hand button

Height Adjustment (4)

Up: Press left hand button
Down: Press right hand button

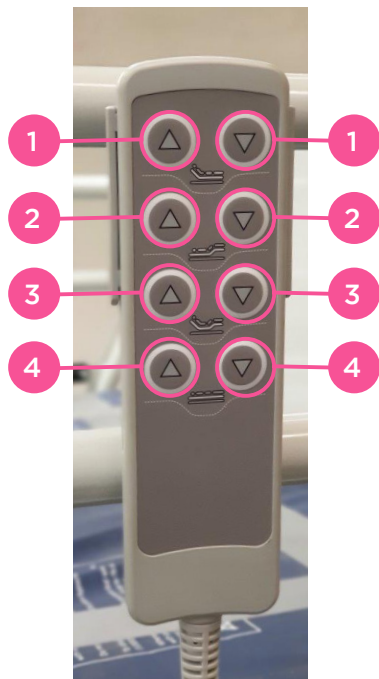

CAUTION!

The bed is intended to be operated by or in the presence of a fully trained medical professional and should only be moved within the patient room for cleaning or patient access but is not intended for patient transportation.



CAUTION!

Ensure the handset is only operated when the patient is in the intended position as shown in the Figure 1.

Ensure the leg support section is flat before the patient enters or exits the bed as shown in Figure 2.



Figure 1



Figure 2

Operators should read full user manual instructions before use.
Scan here for user manual.

